SKIN  TECH

Instructions
for use

Easy TCA[®]:

Starter kit 15%

Classic kit 15%

ONE LIFE | ONE SKIN

1014

The Medical Device Class IIa **Easy TCA**® is designed for peelings, depth 3/7 or 4/7 or 5/7, phototype FZ 1-6. See table depths of peelings (from 1/7 up to 7/7) in the “General Information Peelings”. It can also be applied out of the face.

PRELIMINARY INFORMATION

TCA: Trichloroacetic Acid.

This product is for the exclusive use by medical specialists in plastic aesthetic or restorative surgery, by specialists in medical and surgical dermatology and venereology, and M.D.s with postgraduate training in aesthetics.

PRELIMINARY PRECAUTIONS

Before starting the use of TCA peelings, read carefully the “General Information Peelings”. The information contained in the records is given as an indication, it helps the doctor to select the proper peeling and the best way of applying it and, consequently, the doctor retains full responsibility for the decision and the act committed. The deeper is the peeling and the higher is the risk of complications.

PRESENTATIONS

- **Easy TCA® Starter:** kit for 4 peelings.
- **Easy TCA® Classic 12:** kit for 12 peelings.
- **Easy TCA® Classic 24:** kit for 24 peelings.

KIT DESCRIPTION

The number of elements varies and is adapted to the number of peelings provided.

- **Base Solution:** Solution for diluting TCA to a concentration of a 15% w/w.
Starter kit: 4 ampoules of 2 mL; Classic 12: 12 ampoules of 2 mL; Classic 24: 2 vials of 14 mL.
- **Aktivator TCA® 50%:** Solution containing a 50% w/w TCA solution.
Starter kit: 1 vial of 3,4 mL; Classic 12: 1 vial of 10 mL; Classic 24: 1 vial of 10 mL.
- **Post Peel Mask:** Cream to be applied after the peeling.
Starter kit: 1 tube 10 mL; Classic 12: 3 tubes 10 mL; Classic 24: 1 tube 50 mL.
- **Divers:** One or more dosing cards for “Skin Tech® Post Peel Mask”, a small container for mixing the TCA with “Base Solution”, device to help breaking the ampoules, cotton buds and instructions for use.
- **Traceability Stickers:** Starter kit: 4 pairs; Classic 12: 12 pairs; Classic 24: 24 pairs.

INGREDIENTS

- **Post Peel Mask:** Aqua (Water), Glycerin, Magnesium Ascorbyl Phosphate, C12-15 Alkyl Benzoate, Cetearyl Alcohol, Caprylic/Capric Triglyceride, Glycyrrhetic Acid, Cetyl Alcohol, Biosaccharide Gum-2, Cyclopentasiloxane, Dimethicone, Methylsilanol Mannuronate, PEG-100 Stearate, Glyceryl Stearate, Potassium Cetyl Phosphate, Acrylamide/Sodium Acrylate Copolymer, Cetearth-20, Sodium Acrylate/Sodium Acryldimethyl Taurate Copolymer, Potassium Phosphate, Polysorbate 20, Tocopheryl Acetate, Disiloxane, Isohexadecane, Paraffinum Liquidum, Phytic Acid, Polysorbate 80, Caprylyl Glycol, Superoxide Dismutase, Glycine Soja Oil, Hydrogenated Lecithin, Sh-Polypeptide-1, Cyclomethicone, Hydroxypropyl Cyclodextrin, Sodium Hydroxide, Chlorphenesin, Urea, Allantoin, C12-15 Alcohols, Trideceth-6, Dimethiconol, Sodium Oleate, Biotin, Retinol,

Propylene Glycol, Dicalcium Phosphate, BHT, Ascorbyl Palmitate, Citric Acid, Phenoxyethanol, Sorbic Acid, L-Selenomethionine, Disodium EDTA, CI 15985, Parfum, Hexyl Cinnamal, Linalool, Limonene, Hydroxycitronellal, Citral, Citronellol.

INDICATIONS

Easy TCA® is primarily intended for a peeling until the basal layer of the epidermis, but it can reach the Grenz zone or the papillary dermis, depending on the application process and the number of coats applied. The indications are: photo-aging and its consequences, acne and post acne, pigmentation problems (melasma, shloasma, post inflammatory hyperpigmentation), ichthyosis, keratoses and lentigine.

CONTRAINDICATIONS

See the general information "General Information Peelings". In particular, the application of Easy TCA® must be cautious and gradual to any patient who has had a previous or current treatment that might increase skin permeability.

SKIN PREPARATION DURING THE WEEKS PRIOR TO PEELING

The skin preparation is optional before Easy TCA® (depths 3/7 to 4/7). Anti-pigment preparation is recommended for deeper protocols or higher phototype (FZ 5-7): apply "Skin Tech® Blending Bleaching Cream" twice daily for 3-4 weeks prior to a peeling depth 4/7 or 5/7 (see "General Information Peelings"). From the level of peeling depth 5/7: Prevention of classical herpes to patients with history.

SKIN PREPARATION IMMEDIATELY BEFORE PEELING

Avoid degrease the skin and/or disinfect with alcohol. In case of thick makeup, the skin can be cleansed with the foam "Skin Tech® Cleanser".

PREPARING THE PEELING SOLUTION

- **Easy TCA® Starter kit and Easy TCA® Classic kit 12:** Open one ampoule of "Base Solution" using the ampoules breaking device. Pour the contents into the container of the kit. Syringe out 0,7 mL of "Aktivator TCA® 50%" and mix with the "Base Solution".
- **Easy TCA® Classic kit 24:** Syringe out 4,8 mL of "Aktivator TCA® 50%" and inject it into a vial of "Base Solution".

Please remember that injecting any volume into a container that is hermetically closed will induce an overpressure that could spray the acid on yourself or on your patient at the moment of removing the needle out of the vial.

PREPARATION OF THE CREAM "SKIN TECH® POST PEEL MASK"


Prepare required amount of "Skin Tech® Post Peel Mask", referring to the dosing card of the kit. One "line" corresponds to +/- 0,5 g of "Skin Tech® Post Peel Mask".

PEELING APPLICATION

Easy TCA® is usually applied with circular movements, using the cotton buds (called "the applicators") included in the kit.

1. APPLICATION OF THE FIRST LAYER

Divide the face in 7 different areas (see the graph below). Soak the applicators into the peeling solution, spin them dry on the edge of the container in order to eliminate an eventual excess of solution and apply on the first area using regular circular movements; repeat this process for each area. Let this first layer dry completely and check "the result".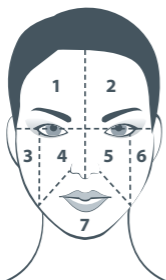

"The result" is not a volumetric notion (milliliters) or a temporal concept (minutes) but rather a clinical sign:

- Erythema signs peeling depth 2/7 (intraepidermal).
- Frosting points sign peeling depth 3/7 (basal layer).
- Frosting clouds sign peeling depth 4/7 (Grenz zone).


If the desired frosting was obtained with the first layer, apply immediately the right amount of "Skin Tech® Post Peel Mask" on the treated skin (see dosing card in kit).

If the desired frosting has not been obtained with the first layer, reapply the Easy TCA® solution on the face, as described below.

2. APPLICATION OF POSSIBLE FOLLOWING LAYERS

Divide the face in 3 different areas (frontal and lateral areas: see the graph on the left) and apply the peeling solution using circular movements. Let this second layer dry and check "the result" (see above). Repeat if necessary the application (divide the face in three different areas) leaving each layer dry, until obtaining the needed frosting.

The "clinical sign" corresponds to the achieved depth (rash or spots or clouds), then we consider that it is the "end of the peeling". At this time, apply the right amount of "Skin Tech® Post Peel Mask" (see dosing card).

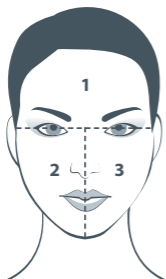APPLICATION OF THE "SKIN TECH® POST PEEL MASK"

Apply the cream "Skin Tech® Post Peel Mask" in the amount determined by the dosing card of the kit. The "Skin Tech® Post Peel Mask" stops pain, erythema, swelling and heat of the skin in a few seconds, and the risk of side effects is drastically limited.

IMMEDIATE POST-PEELING

Cream "Skin Tech® Post Peel Mask" remains on the treated area for at least 8 hours before being washed with cleansing foam "Skin Tech® Cleanser" which balances the pH of the skin to its natural level (pH 5,5).

PREDICTABLE EFFECTS DURING THE FIRST WEEK POST PEELING

Dry skin begins to peel on the third day (sometimes later in case of first peeling) and finishes until the sixth day. The desquamation intensity depends on the depth of the peeling.

- **Depth 2/7 (intra-epidermal):** desquamation almost invisible.
- **Depth 3/7 (basal layer):** desquamation 3 days like sunburn.
- **Depth 4/7 (Grenz zone):** the skin may darken in peeling zones 4/7 and it results in a more visible peeling.

DAILY CARE

Starting from the day following any peeling, sun protection: Skin Tech® Melablock-HSP® SPF 30 or 50+ is required throughout the treatment and until the end of the second week after the last peeling (applications at 9am, 1pm, 3pm).

From the day after the first peeling, the patient shall apply once or twice a day the creams of the line Skin Tech® described below, until the disappearance of any problem.

- **In case of acne:** PURIFYING Cream (between peeling sessions) or PURIGEL Gel (long term treatment).
- **In the case of pigmentation:** BLENDING BLEACHING CREAM.
- **In case of dry skin:** VIT E Anti-oxydant Cream or NUTRITIVE CREAM Vit. A-C-E Lipoic Complex.
- **In the case of aging:** DHEA PHYTO Cream or ATROFILLIN® Cream.
- **In case of skin laxity:** ACTILIFT® Cream.
- **In the case of post-peeling erythema:** IPLASE Mask®.
- **In the case of age related skin atrophy:** ATROFILLIN® Cream.

Generally, if any cream produces a sensation of intense heat, suspend the application for a few days.

REPETITION OF PEELING

Easy TCA® is usually repeated 4 times, (3-5) according to the physician's decision. The repetition frequency is one session a week on facial skin, when it is extra-facial skin one session every two weeks. 4 sessions of peeling constitute total treatment that can be repeated after a minimum of 6 weeks of rest of the skin.

Note: Application of Easy TCA® in hands, forearms and neck

The protocol to be used is different: apply a first layer of peeling solution and let dry. Apply a second layer and let dry. When the skin is dry, verify the presence or absence of frosting points. In the presence of frosting points, apply the cream "Skin Tech® Post Peel Mask". In absence of frosting points, apply a third and final layer of Easy TCA®, let dry and apply "Skin Tech® Post Peel Mask". Repeat 4 times, once every two weeks.

PRECAUTIONS

Avoid direct sun exposure during the treatment periods. Do not apply on the damaged or infected skin, or on open wounds or on recent scars. Do not rub or pull the desquamated skin. Wash hands before and after use. Do not inhale; do not allow the peeling solution to enter the body by any means whatsoever, other than the application on the skin. Do not use on wet skin (dilution of active ingredients) or dirty skin (the organic substances present on the skin could reduce the action of the acids). Avoid contact with eyes: In case of contact with eyes, an intense burning sensation appears: Rinse immediately with a saline solution until the burning sensation disappears. Consult with a specialist if the solution comes in contact with eyes. If swallowed, take appropriate protective measures, consult a physician immediately and show the container or label. If necessary, contact with a toxicological centre nearest you. Store between +5 °C and +35 °C. Do not use if the packaging (box or bottle) has been damaged.

Keep out of the reach of children. Not suitable for preteens.

POSSIBLE SECONDARY EFFECTS

Patient could suffer side effects during and after the peeling such as:

- (Minor, disappear in 24h) burning sensation / Itching / Erythema / Swelling
- (Not common, please contact the doctor who treated you in any case) Allergic reaction / Ulceration / Hyper or hypo-pigmentation / Scarring / Secondary infection / Cardiac arrhythmia...

The doctor is requested to report any notable side effect related to the medical device Easy TCA® to Skin Tech Pharma Group SL. through the website www.skintechpharmagroup.com, by e-mail at info@skintechpharmagroup.com or by post.

DISPOSAL

Comply with local, national and European regulations for the disposal of wastes and toxic chemical hazards of contaminated wastes.

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